




Attachment of Biosafe 4-point harness on Ferno POWERX & CCT-PX Trolleys

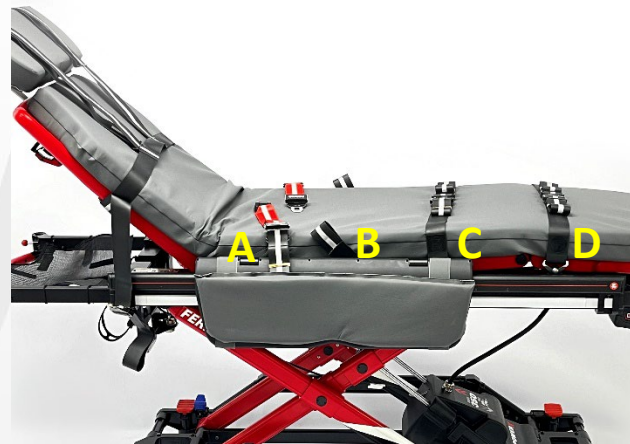

Position the upper-body harness on the stretcher as per above photo, placing the harness on top of the mattress and attaching the cross-strap of the harness (1) to the lower fixing points on the back of the backrest as shown in the second photo.



- Biosafe 4-point harness with built in chest-strap for POWERX, p/n 038387550
- Biosafe loop strap for attachment of PediMate/NeoMate, p/n 038247650
- Biosafe leg strap for POWERX, p/n 038387600 (2 used)

Ensure the upper-body harness waist straps (A) and leg straps (C & D) are attached to the side of the stretcher at the fixing points as per above photo. Note that the tangs can be a little difficult to attach to the fixing points as there is not a lot of room between the fixing point “mushroom” and the sides of the SX cot sides.

Loop strap for attachment of PediMate (B) should be attached to the fixing point located between waist strap (A) and first leg strap (C).



Deployment of Biosafe 4-point harness on Ferno POWERX & CCT-PX


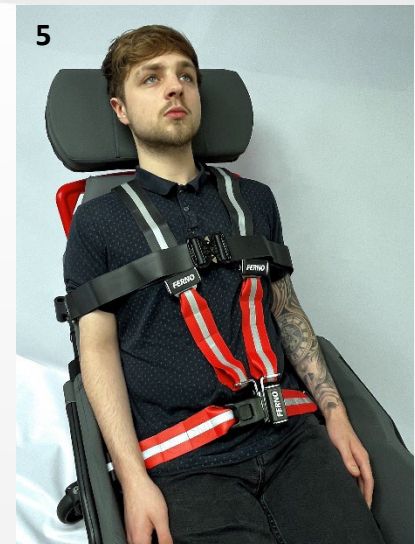
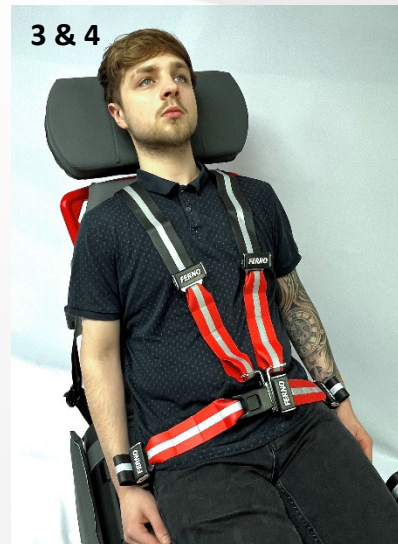


Deployment:

- 1 First ensure that the chest strap is un-fastened and the shoulder straps unbuckled and laid over the back of the backrest or over the headrest.
- 2 Shoulder straps go around the patient from behind the back, up over the shoulders.
- 3 Attach together at the front to the waist straps, feeding the T-bars through the tang on the end of one of the waist straps and then connect to the buckle.
- 4 Adjust all strap lengths (shoulders and waist) so patient is held securely but comfortably in the harness. Waist straps should be correctly positioned around waist.
- 5 Deploy the chest strap around the patient's upper body and tightened/adjusted to the required length such that the patient is held securely but comfortably.



Attachment PediMate PLUS / NeoMate on Ferno POWERX & CCT-PX Trolleys

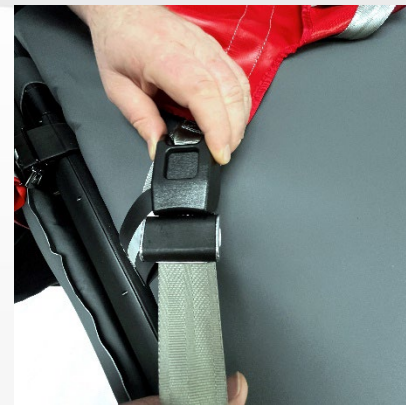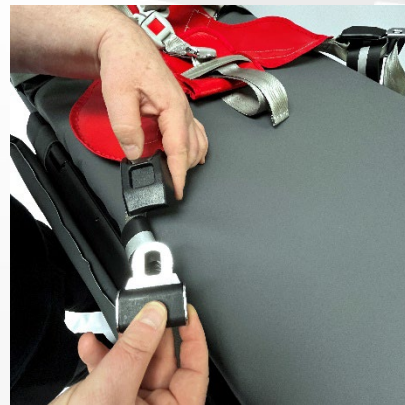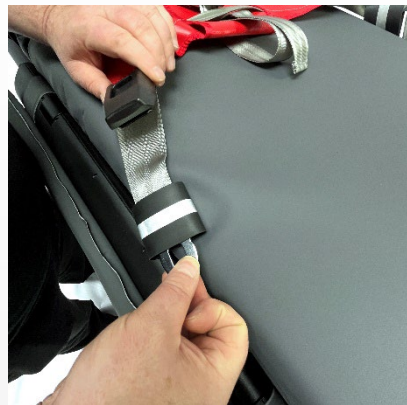

- POWERX does not have a tubular side-frame to attach PediMate PLUS lower straps to, like on Pegasus, Megabus or Harrier.
- When using POWERX 4-point harness there is a dedicated loop strap that is attached to the side of the stretcher for attaching the PediMate lower straps through.
- This loop strap (p/n 038247650) is attached to the side of the stretcher as per above photo, position B.
- The upper straps of PediMate PLUS are attached around the backrest as normal.
- Neomate attaches to POWERX in the same way as PediMate PLUS, but with 1 upper strap attaching around the back of the backrest, compared to the 2 upper straps that are on PediMate PLUS.