

Power-PRO™TL

Quick Start Guide*

Power-PRO™TL is an electric, hydraulic ambulance stretcher, designed to help reduce the risk of manual handling injury with powered lifting and features that make it easy to manoeuvre and fit into small spaces.



Main Controls


1. Foot end button to raise/lower or extend/retract frame
2. Corner poles for pushing and steering – button release
3. Red trigger buttons to telescope head and foot in/out (both ends)
4. Trigger handle to raise/lower back rest
5. Squeeze to release side rail
6. Loops to raise knee gatch
7. Release lever for knee gatch and shock position
8. Caster steer lock switch
9. Battery
10. Brakes (two or all four wheels)

*This Quick Start Guide is not intended to replace the manufacturer's Operations Manual or product training.

Always read the manual before use for full instructions and safety warnings. © Stryker 17 Jul 08

Power-PRO™TL Quick Start Guide*

Frequently Asked Questions

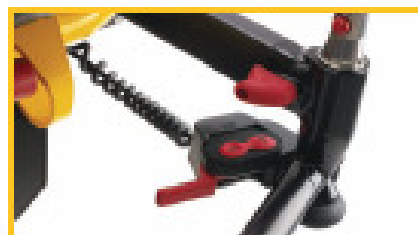

How long does the battery last?

Will last around 20 complete calls (at least 2 shifts)

Red warning light when 3 calls left

Simple swap out

If needed, manual override with safety cut-out

How long does it take to charge?

About an hour. When the red light on the cot flashes, just swap the batteries over. Leave the drained battery on the charger until it's needed again.

(See full battery management guidelines for further details).

What is the cot like to lift?

Similar weight to competitive cots (66kg)

Fixed hydraulic system – wheels don't drop so easier to lift

Can it be power washed?

Yes, just take the battery off first

Can you raise the cot in the ambulance?

No. It needs to be in its lowest position to fit in the locks.



*This Quick Start Guide is not intended to replace the manufacturer's Operations Manual or product training.

Always read the manual before use for full instructions and safety warnings. © Stryker Jul 08 Lit ref PHE0801